




ARROWHEAD MEDICAL RESOURCES

www.arrowheadmed.com

(888) 932-0016 Toll Free (218) 328-0016 Office (218) 328-0015 Fax

All Arrowhead Medical electrotherapy electrodes provide consistent and reliable electrical dispersion across the surface of the electrode.

Our silver-coated carbon provides you with an electrode that utilizes its full surface area for your electrotherapy treatments.

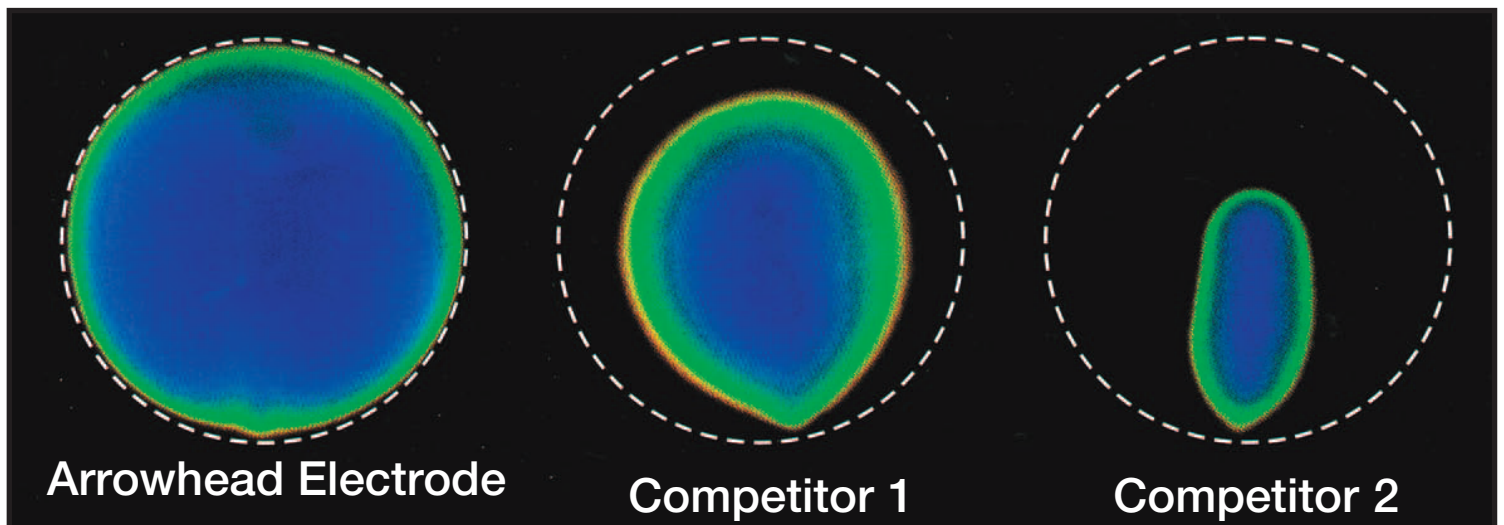

This statement is demonstrated by the following test:

1. Three different 2" dia. prewired electrodes (Arrowhead Medical electrode, Competitor 1, and Competitor 2) are placed upon a conductive heat reactive, liquid crystal material.
2. A direct current is passed simultaneously through each electrode for a period of 10 seconds.
3. Results observed:
 - Arrowhead Medical electrode, as evidenced by the heat activated color change, distributes the electrical current across the full surface of the electrode. There are no "hot" spots.
 - Competitor 1 utilizes only about 75% of its surface area.
 - Competitor 2 utilizes only about 25% of its surface area.

All Arrowhead Medical electrodes are built to the highest ISO and FDA quality standards.